

# CDental/MDI Export Instructions – iTero Scanner ONLY

## EXPORTING THE STL FILES FROM ITERO OR INTRA ORAL SCANNER



- Make sure that you have captured the bite scans in correct occlusion. An accurate wax up and planning cannot be done if the bite is not captured correctly. Review scan to check occlusion.
- Make sure that there are no holes or distortion on the teeth. Holes or distortion may result in a guide that does not fit.

**STEP 1**



**LOG IN TO WWW.MYITERO.COM WITH YOUR PRACTICE LOGIN AND PASSWORD** *Or other intra oral scanner software.*

**STEP 2**



**SEARCH FOR THE PATIENT NAME**

**STEP 3**



**CLICK ON THE PATIENT NAME TO OPEN THE CASE FILE**

**STEP 4**



**CHOOSE EXPORT**

**STEP 5**



**CHOOSE THE CORRECT 3 OPTIONS SHOWN IN THE IMAGE BELOW (OPEN SHELL, FILE PER ARCH ORIENTED IN OCCLUSION, STL + PLY)**

Orders			
ID	Scan Date	Case Type	Status
111019404	09/11/2022	Invisalign + IRecord	Completed

View Rx   Change Case Type   Viewer   **Export**   iTero Scan Report   Invisalign Outcome Simulator

Invisalign Progress Assessment   Invisalign Go Case Assessment   Invisalign Go Outcome Simulator   OrthoCAD

Export Type   Open Shell

Data Format   File per Arch (arches oriented in occl...)

File Type   STL+PLY

Hide patient name

Cancel   Export

### UPLOADING THE ITERO SCAN TO C-DENTAL/MDI FOR MERGE WITH A CBCT TAKEN AT C-DENTAL OR MDI

STEP 1

**FILL OUT THE UPLOAD FORM ON THE [WWW.CDENTAL.COM](http://WWW.CDENTAL.COM) WEBSITE**

[Click Here for Upload Page](#)

STEP 2

**WAX UPS ARE ENCOURAGED FOR ACCURATE PLANNING. WAX UPS ARE \$25 PER TOOTH. BE SURE TO INCLUDE WAX UP SITES FOR MISSING TEETH IF NEEDED**

STEP 3

**CHOOSE ONE OF THE FOLLOWING OPTIONS**

WAX-UP : Virtual Wax-Up - CBCT + Intraoral Scan (indicated to...)

NON WAX-UP : Merge DICOM & STL Only

STEP 4

**BROWSE TO THE DOWNLOAD FOLDER AND ATTACH THE ORTHOCAD EXPORT FOLDER THAT YOU EXPORTED FROM [WWW.MYITERO.COM](http://WWW.MYITERO.COM)**

*See page 1 for export instructions.*

Attachment - Please zip STL files into one folder and use the patient name as the title of the folder\*

Drag and drop files here

Browse

STEP 5

**MAKE SURE TO SELECT THE ONE WITH THE DATE AND TIME THAT YOU EXPORTED THE FILE**

> This PC > Downloads

Name

Date modified

Today

OrthoCAD\_Export\_111019404

11/9/2022 10:07 AM

STEP 6

**Make sure to HIT SUBMIT**



If you have any questions, please contact our Sara Tarazi - Chief Technologist

✉ [starazi@cdental.com](mailto:starazi@cdental.com)

☎ 510-368-7240